

AEOLUS


Professional Vet Equipments Supplier




AEOLUS

Icu/ Incubator Organizing Rack


UC-1801-RCK

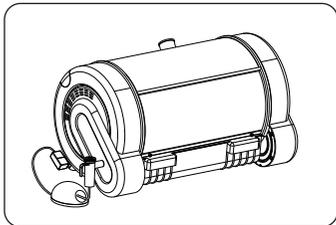

UC-1803

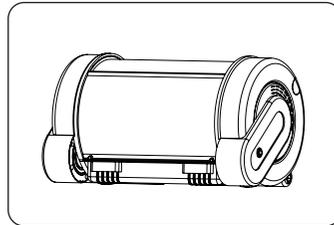

UC-1801

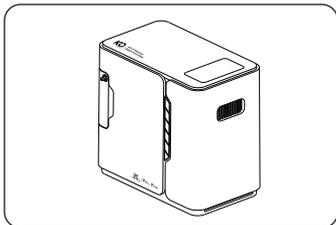

KO₂-2F



KO₂-10F

Distributor Contacts


DALLAS

11307 Indian Trail, Dallas, Texas 75229, USA.
Tel: +1 469 930 6359

SHANGHAI

Floor 5, Building 6, No. 675, Zhujin Road, Songjiang District, Shanghai, China.
Tel: +86-21-67632060, 67632364

New
Generation

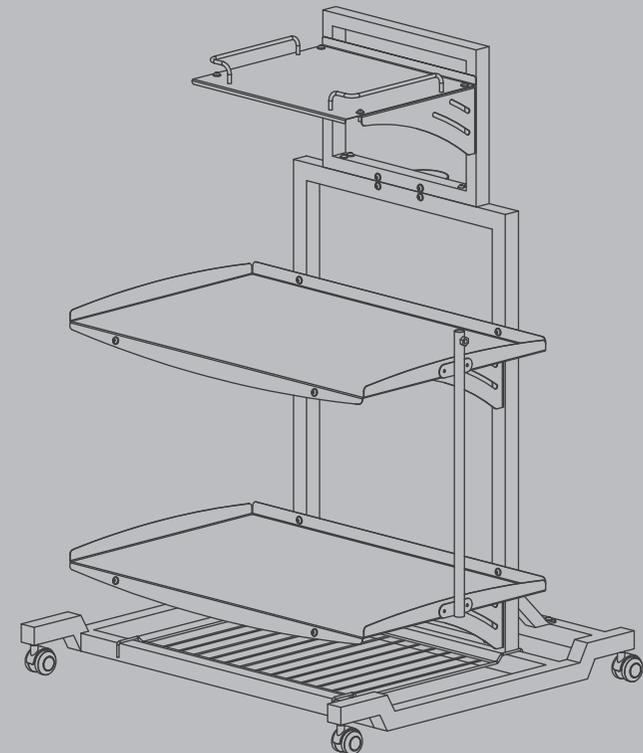

User Manual

Brief Introduction

Double decker design, well for two units of KaniKare UC-1801 ICUs.
Specialized top shelf for air conditioner or oxygen generator locating.
Stable and Sturdy, made from Gauge 17 steel panel, ensures long service life.
Comes standard with casters, for easy moving and sterilization operation.
Trendy looking but functional too, a good mate for ICU accessories organizing.

Package Inspection

The manufacturer is not liable for any damage caused by a third party "Transport or Delivery Company".

The day the unit is delivered, an inspection of the outer packing should be made for any damage to the packing box.

Then the unit should be unpacked carefully; all components, accessories and the table itself should be inspected for any damage or loss of component parts.


If any damage is evident, the carrier or dealer should be notified immediately.

Safety Warnings


1. Use only as described in this manual. Use only manufacturer's recommended attachments.
2. This product is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the product by a person responsible for their safety.
Children should be supervised to ensure that they do not play with the product.
3. DO NOT modify, disassemble or repair.
4. DO NOT use outdoors or on wet surfaces. DO NOT place the product on an uneven or soft ground.
5. Avoid using the product in dusty, vibrating and wet environment and in chemical storage place.

Assembly Instructions


1. Screw four casters to base.

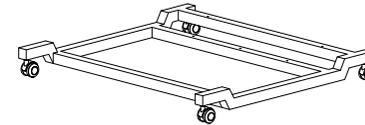


2. And then install a big rectangular frame with 3 screws (M8*60).

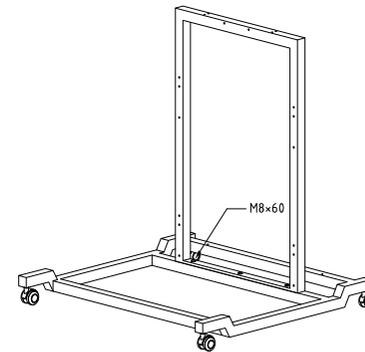


3. Install a small rectangular frame onto the top of the big frame with screws (M8*95).

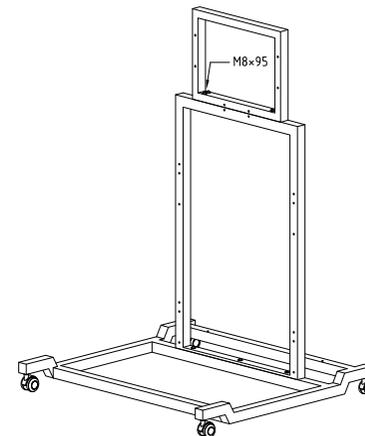


4. And fix the semi-circle part with screws (M8*55) onto the position shown in the picture.

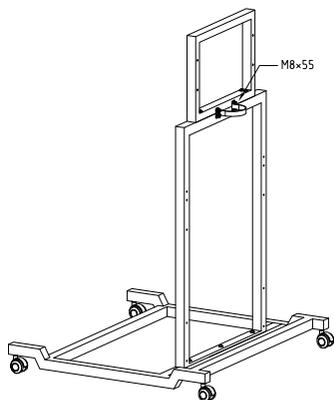

5. Fix the auxiliary connectors to the base with screws (M8*55)

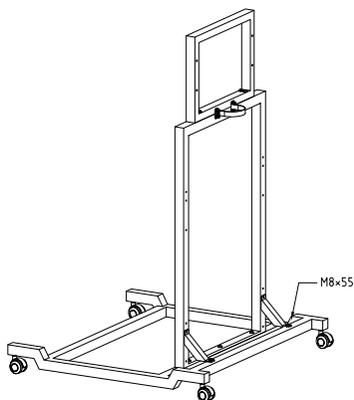


6. Put the bottom tray-holder on the front. And then align the holes of the tray-holder, the big rectangular frame and the auxiliary connector. Fix them with screws (M8*55).

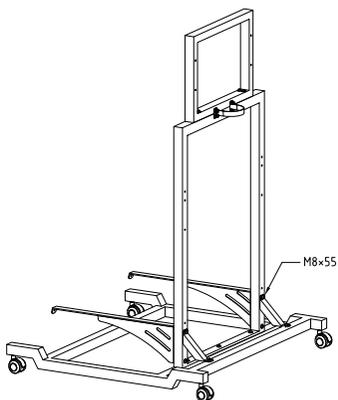

03


7. Align the holes of the middle tray-holder and the big rectangular frame. Fix them with screws (M8*55).

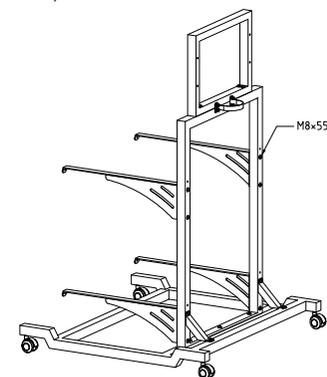

8. Align the holes of the top tray-holder and the small rectangular frame. Fix them with screws (M8*55).




9. Put the bottom tray on the holder, align the holes and fix with screws (M8*55 and M8*20). And then install the middle holder/tray and the top holder/tray.

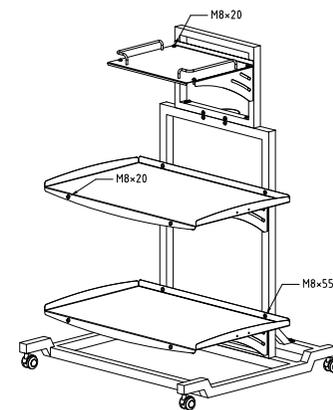


04

10. Put the IV-pole on the side of the trays, align the holes and fix with screws (M5*15).

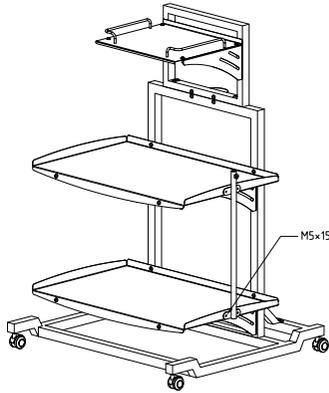

11. Clip the shelf in the base.

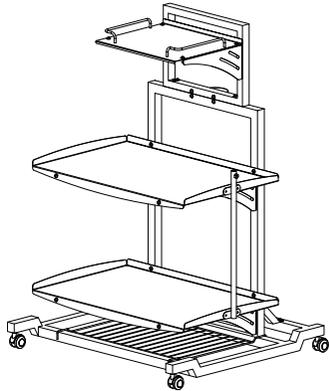

Maintenance

In between usages, the rack should be cleaned with a soft cloth or sponge using a mild cleaning agent (and then disinfected).

Service Support

Contact us if you cannot solve your problem with this manual.

Dallas

11307 Indian Trail, Dallas, Texas 75229, USA.

Tel: +1 469 930 6359

Web: www.aeoluspet.com

Sydney

147 Dwyer Road, Leppington NSW 2179, Australia

Tel: +61 2 9606 4520

Singapore

150 South Bridge Road, Singapore 058727

Tel: + 65-6533 6524 / 9633 2734

Shanghai

Floor 5, Building 6, No. 675, Zhujin Road, Songjiang District, Shanghai, China.

Tel: +86-21-67632060, 67632364