


# AEOLUS

Professional Vet Equipments Supplier




## PET ICU.INCUBATOR


UC-1801

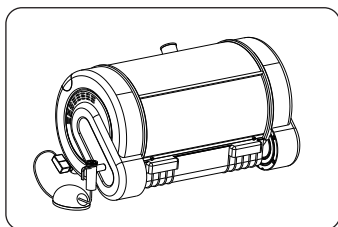


UC-1803

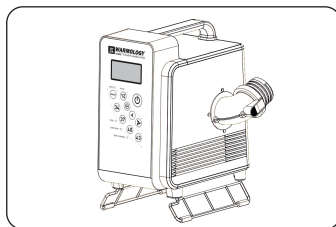

WMS-1501

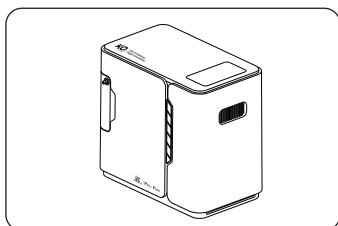

KO<sub>2</sub>-2F

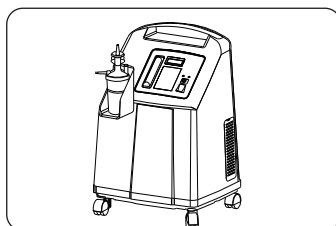

KO<sub>2</sub>-10F


Distributor Contacts

### **DALLAS**

11307 Indian Trail, Dallas, Texas 75229, USA.  
Tel: +1 469 930 6359

### **SHANGHAI**

Floor 5, Building 6, No. 675, Zhujin Road, Songjiang District, Shanghai, China.  
Tel: +86-21-67632060, 67632364

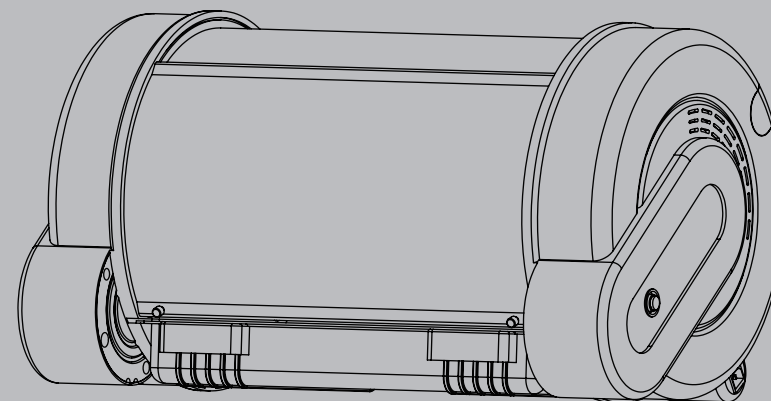

## User Manual

# CONTENTS

Overview	01
Summary of Important Notes	02
Product Use	02
Product Installation	02
Product Principles and Functions	05
Functions of the Product	06
Product Operation Instructions	09
Cleaning and Maintenance	14
Troubleshooting	17

Thank you for using our AEOLUS UC-1801ICU.INCUBATOR.  
We want you to get the most out of the machine and to have your experience in using it proceed with ease. Detailed instructions are provided along with the product's introduction, usage and to correctly use it and take full advantage of all operations and functions.  
In case of any printing or translation error, we apologize for the inconvenience. As for the content change, we are sorry for no further notice.

## Overview

The Instruction Manual is used for AEOLUS UC-1801ICU.INCUBATOR (Pet ICU/Young Pet Incubator) designed by AEOLUS International.

Installation, use, cleaning, maintenance and repair of the ICU, as well as principles and functions of the product, caution and warnings, common fault analysis and troubleshooting and after-sales services etc. are all in the Overview. Users should operate the product in accordance with requirements as stated in the Instruction Manual. The warranty does not apply in cases where the product fails due to incorrect operation or improper maintenance by non-professionals. All staff use the equipment shall read the Instruction Manual carefully before turning on the equipment. If you have difficulty in understanding the Manual, please contact a local AEOLUS product distributor or the company for more detailed information.

AEOLUS UC-1801-ICU.INCUBATOR (Pet ICU/Young Pet Incubator) is a newly developed pet ICU and young pet incubator developed to serve an enormous need and fill a vacancy in the domestic pet care industry. This moderately sized piece of pet medical equipment will easily fit into most veterinarians' practices and pet hospitals. Its characteristics and compact structural design functions perfectly serve its purpose. The mechanism adopts "dual duct and double circulation convective thermal regulation" to create an excellent and bacteria-free therapeutic environment and young pet incubation environment similar to that provided pet parent (accompanied by purified air and proper temperature and humidity control).

In many countries, our Company is the first to introduce computer technology and auxiliary medical devices for pets in the area of ICU control systems. This allows for more accurate temperature control of ICU, and brings greater convenience for clinical use. The machine is equipped with an intelligent alarm device for five faults sensor, extreme temperature, excessive moisture, indoor carbon dioxide concentration, lack-of-water in the tank and power supply Interruption). This alarm system is tested and proven to be safe and reliable. Furthermore, it is equipped with a unique cabinet structure and double-layer thermostat cover, which can prevent loss of heat and radiant heat when nursing the pets. In addition, a temperature and humidity display device is provided, measuring current temperature with the set temperature clearly visible. The heating control adopts dual duct and double circulation convection heating technology. This prevents rapid change in temperature minimizing the shock that can be caused. It ensures that temperature of the ICU can rise evenly and gently. The Thermostat System adopts PID control technology, which can solve the problem of inertia temperature error comprehensively, thus the temperature control is more accurate and more energy efficient.

AEOLUS UC-1801 ICU.INCUBATOR is a fully integrated pet ICU and incubator.

AEOLUS UC-1801 ICU.INCUBATOR accurately controls temperature and humidity. It is the only domestic product with balanced heating and stable temperature and humidity (used in ICU).

AEOLUS UC-1801 ICU.INCUBATOR provides a bacteria-free therapeutic environment, which can efficiently kill all harmful bacteria and viruses.

AEOLUS UC-1801 ICU.INCUBATOR incorporates integrated adjuvant therapeutic functions, not just the functions of an ICU.

AEOLUS UC-1801 ICU.INCUBATOR provides air purification with medical effects, rather than concept speculation

AEOLUS UC-1801 ICU.INCUBATOR accurately monitors the medial environment, which safeguards the pets' lives.

AEOLUS UC-1801 ICU.INCUBATOR is an Earth friendly product; saves energy and protects the environment, which can effectively savings 35% of energy (when compared to other pet care products).

AEOLUS UC-1801 ICU.INCUBATOR especially provides the creation of a bacteria-free environment. The dual duct and double circulation convection heating technology and the PID thermostat control technology are pioneering, which ensure the reliability, safety and practicality of the machine.

# Summary of Important Notes

1. Please read the Instruction Manual carefully before using the machine. Nurses (and other pet care workers) should be trained to operate the machine under the guidance of qualified medical care personnel who are familiar with the functions of ICU/Young Pet Incubator and understand the risks and advantages of the machine.
2. Please count the parts and spare parts when unpacking, and read the Instruction Manual carefully.
3. The equipment shall be placed in a clean place with limited changes in temperature and humidity, protected from direct sunlight and away from heat radiation.
4. To ensure the proper operation of the Pet ICU, please DO NOT fill the bottle with distilled water or purified water.
5. After the machine is turned on, please preset the temperature value, and warm it up for 3-5 minutes before use.
6. Please do not load weight onto the machine, and the maximum load capacity of the machine housing is 1-2 kilograms (2.2 to 4.45lbs).
7. Unsanitary cleaning materials will cause the generation of carbon dioxide, and thus increase the carbon dioxide concentration in the pet ICU. The equipment should be inspected and cleaned. And the air purifying materials shall be replaced in a timely manner and on a regular schedule.
8. The machine is equipped with an intelligent alarm system sensor for five faults: extreme temperature, excessive moisture, indoor carbon dioxide concentration, lack-of-water in the tank and power supply Interruption. This alarm system is tested and proven to be safe and reliable. If the machine fails, please inspect the systems immediately to find out reasons and eliminate the faults. If the faults cannot be eliminated in a short time, please stop medical treatment or care for pets, and ask for professionals or personnel from the AEOLUS Company to repair the machine.
9. Allow for the machine to have proper ventilation. Do not block the air outlet and return air inlet.
10. After being powered on, if the machine is not used, in order to ensure safety and energy-saving, please turn the switch to off position and allow for sufficient power down before turning the switch back to on position.
11. Do not use alcohol or other organic solvents to wipe the thermostat cover of the tank. Keep it moisture free.

## Product Use

1. Provides bacteria free and thermostatic incubation cocoon for young pets.
2. Allows for body temperature recovery, fluid infusion, atomization treatment, incubation, health improving and comfortable, stable stay in the hospital (for observation for sick, dying and weak pets).

## Product Installation

Built-in humidifier bottle: some parts of medical atomizer and the tank are separately packed. Please pay attention not to damage or miss any parts or spare parts when unpacking.

### 1. Installation Checklist of the Whole Machine:

AEOLUS UC-1801 ICU.INCUBATOR cabinet	1
Built-in humidifier bottle	2
Plastic neck of the humidifier bottle	3
Main body of the medical atomizer	4
Nozzle of the medical atomizer	5
Medicine cup of the medial atomizer	6
Water tank of the medical atomizer	7

Soft gasket of the medical atomizer	8
Power line of AEOLUS UC-1801ICU.INCUBATOR	9
Stainless steel tray of AEOLUS UC-1801ICU.INCUBATOR	10

### 2. Installation of Humidifier Vessel

As shown in Figure 4.2.1, the Humidifier Vessel is installed in the right control chamber of the AEOLUS UC-1801 ICU.INCUBATOR. As shown in Figure 4.2.2, open the outlet cover, take out the built-in Water Vessel, fill the built-in Water Vessel with water, (exclusive of distilled water or purified water) as shown in Figure 4.2.2. And then put back the vessel and place the cover in the proper position.

#### Notes:

- a) The Humidifier and Water Level Sensor should be placed securely onto the vessel to prevent the spill of water.
- b) Do not fill the vessel with purified or distilled water, which may cause the malfunction of the Water Level Control of the water tank and may eventually cause water to spill or leak.
- c) The water level should not be above the Highest Level Line (HLV).
- d) The purpose of the water purification filter material is to remove hard water scale and improve the water quality. The validity period of each placement is 10-12 months. Colorless, tasteless and non-toxic, the filter material meets the National Sanitary Standard for Drinking Water & GB1499- 1988 and can effectively improve water quality.

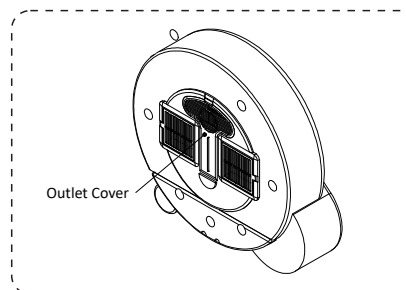


Fig.4.2.1

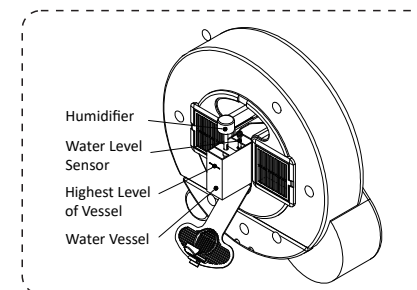


Fig.4.2.2

### 3. Nebulizer Parts Installation

The 850EA Nebulizer is installed at the left side of AEOLUS UC-1801-ICU.INCUBATOR, could be exposed once the protection grid removed, as showed in figure 9.1.1.

Five parts compose the 850EA nebulizer, including nozzle, cup lid, nebulizer panel, medicine cup and air pipe.

These 5 parts need to be assembled in the order indicated in figure 9.1.2 and fixed by the C clip (f in figure 9.1.2) before nebulizer switched on.

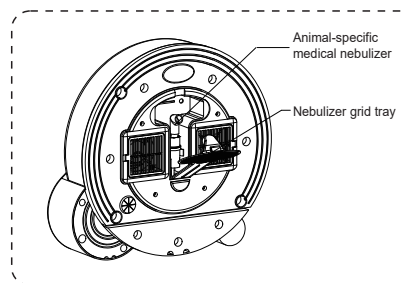


Fig.9.1.1

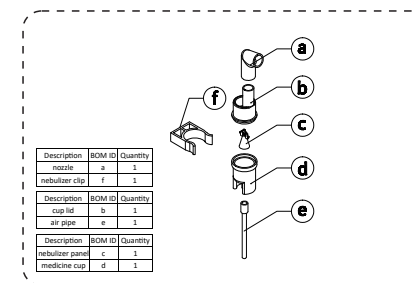


Fig.9.1.2

### Usage Warning:

- The treatment process should not exceed 20 minutes for every nebulizing, misuse would possibly cause Pulmonary edema or water poisoning.
- To prevent superinfection, please pay close attention to the sterilization of nebulizer, ambient air and all the related equipment, also disinfect animal mouth, nose and throat.

### Mechanism of action

Animal respiratory system is an open system, where alveolus is the main air-exchanging place. The object sprayed out of the nebulizer is mainly the mixture of oxygen and medicine – oxygen-medicine aerosol. This kind of aerosol could improve oxygen metabolism and anoxic metabolism significantly; increase the oxygen reserve in organisms. The cavitation effect of compressed air could disperse the liquid medicine into gas phase and turn the liquid into fog form. The suspended particles will get deep into animal respiratory system and treat the focus of infection directly. The feature of air compressing nebulization technique is generating Big Fog Volume Fog with Ultra Small Particle Size (0.5 -5 $\mu$ m); the even fog could go very deep into dendritic bronchus and the particles could settle down in deep respiratory system, achieves painless cure effect. It is a good helper to improve curative effect and accelerate recovery process.

### Tips:

- The smaller the particles are, the better curative effect the nebulization could achieve.
- Normally the fog with bigger particles generated from traditional ultrasonic nebulizer would be easier to be observed, as the fog seems to be thicker.
- The international of human medical nebulizer is the particles in fog need to be smaller than 5 $\mu$ m.

### Specialized Medical Nebulizer for Animal – Better Curative Effect

850EA Air Compressor Nebulizer uses high pressure gas jet knockout the medicine solution into tiny particles and carry them into deep respiratory system. The size of droplet will affect the settlement site of medical fog.

Droplet Size	Settlement Site
1-5 $\mu$ m	Dendritic Bronchus, Fine Bronchus, Alveolus
5-20 $\mu$ m	Bronchus
20-40 $\mu$ m	Upper Respiratory Tract mainly, Nose, Throat, etc.

The patented Energy Saving Low Noise Compressor (less than 25 decibels) generates high speed air jet from filtered clean air (by anti-bacterial cotton), the air stream disperses the medicine solution from liquid cup into tiny particles and form medicine aerosol. The average size of droplet could reach 2.5 $\mu$ m minus which could reach deep respiratory system like alveolus and fine bronchus. The medicine is nebulized evenly, no diluted needed, so there is no residues left almost and could achieve very high medicine utilization rate. This device is preciously made medical equipment with very low faulty rate and very good durability, could serve for 1000 -1500 hours as minimum lifespan.

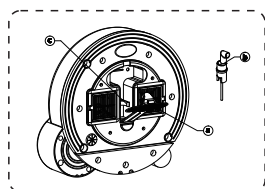


Fig.9.2.1

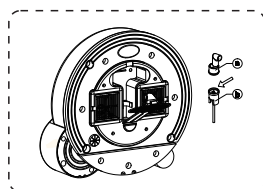


Fig.9.2.2

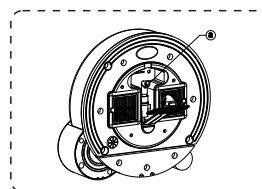


Fig.9.2.3

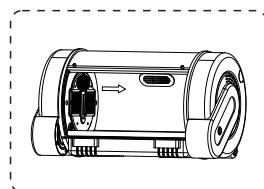


Fig.9.2.4

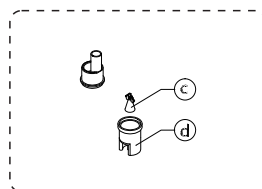


Fig.9.2.5

### Operation Instructions:

850EA nebulizer is easy to operate, please follow the simple steps as follows.

- As showed in Figure 9.2.1, open the nebulizer grid, remove the nebulizer cup from the C clip. If the operator is skilled at this process, he/she could also follow the second step without taking the cup off.
- Figure 9.2.2, screw off the cup lid and nozzle; pour medicine liquid into the nebulizer cup.
- Figure 9.2.3, screw on the cup lid and nozzle, and put them back to the left side nebulizer chamber.
- Figure 9.2.4, Press Nebulize button on control panel to switch on nebulization, the medical fog will be sprayed through grid and get into the ICU chamber. There are two stages of nebulization treatment timing, in which the basic time is 10 minutes. Once the nebulization process started, it is 10 minutes countdown automatically; pressing the "-" button on the nebulize adjustment button will turn it into 20 minutes mode, pressing the "+" will turn back to 10 minutes mode. Pressing Nebulize button again on the main control panel when the nebulizing process is running will switch the nebulizer off.

### Tips:

- There is hissing noise when nebulization starts, that's generated from the high speed air flow. The patented low noise air compressing technology ensures no vibration and no electronic noise when nebulizer runs. The noise is much lower than normal level of most air compressor nebulizer, which is around 85 decibels normally.
- When nebulization is ON, the ventilation fans will be turned off to ensure good nebulizing effects, and humidity and temperature control will be turned off too. The humidity alarm will Not be triggered in nebulization mode, which will reset in 3 minutes after nebulization function turned off.
- The nebulizer panel should be placed at right position as showed in this picture. See Figure 9.2.5

## Product Principles and Functions

### 1. Mechanism of AEOLUS UC-1801 ICU.INCUBATOR

The air circulation system as shown in Figure 5.1.1 is driven by the circulating air of AEOLUS UC-1801 ICU.INCUBATOR (Pet ICU/Young Pet Incubator). After being filtered by the purifier, the air from the outside enters into the ICU, and "dual duct and double circulation convection heating technology" is adopted to create an excellent and bacteria-free ICU therapeutic environment and young pet incubation environment. It provides clean air and proper temperature and humidity.

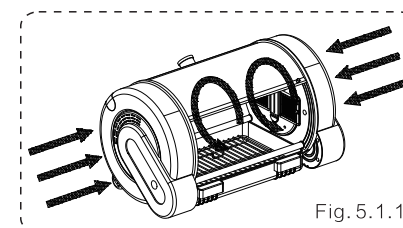


Fig. 5.1.1

### 2. Main Technical Indicators

- Power supply voltage: 220V $\pm$ 10%/50Hz $\pm$ 2%
- Input power:  $\leq$ 400VA
- Ambient temperature: 10°C ~ 35°C
- Temperature control range  
Cabinet temperature: 15°C ~ 38°C (can be up to 39°C by special operations)
- Temperature fluctuation:  $\leq$ 0.8°C
- Average temperature of monitoring chamber:  $\leq$ 1.0°C
- Cabinet temperature control accuracy:  $\pm$ 0.5°C
- Temperature rise time: 5min ~ 20min
- Noises in monitoring room:  $\leq$ 30dB
- Whole machine earth leakage current:  $\leq$ 0.5 mA (normal state)  $\leq$ 1 mA (single fault state)
- Withstand voltage: 1500V/50Hz, lasted one minute without breakdown and flashover.
- Ambient conditions: ① Transportation and storage: a. Ambient temperature: -10°C~40°C  
b. Relative humidity:  $\leq$ 95%  
② Operating conditions: a. Ambient temperature: 18°C ~ 30°C  
b. Relative humidity: 40% ~65%

# Functions of the Product

## 1. Accurate Temperature Control---Being Equal Everywhere

Heating control adopts dual duct and double circulation convection heating technology to allow temperature rising quickly, and temperature shock is small to ensure the temperature of the ICU can raise evenly. Refrigeration control adopts special pet medical refrigerator matching system, which is characterized by fast cooling and obvious effect. Furthermore, dual duct ice can be used for auxiliary cooling. (Note: medical refrigerator is optional)

Thermostat system adopts PID control technology, which can solve the problem of inertia temperature error comprehensively, thus the temperature control is more accurate and more energy-efficient. The humidifier water tank adopts built-in technology, and the integrated design of the product enables the operation more reasonable.

It is easier to clean the scale in the humidifier, just put a few drops of vinegar into the tank, and then the scale can be cleaned.

Silent humidification, even humidification and more accurate control.

Equipped with automatic water cut-off protection function. When the water supply is cut off, there is still certain amount of water left in the tank; thus, it is impossible for idle burning, which is safer to use.

## 2. Accurate Humidly Control---Healthy and Balanced

Without being heated, high-frequency oscillation technology is adopted to produce mist. Compared to heating atomization, it can save 90% of energy. During atomization, large amounts of negative ions are released, and have static electricity reaction with smoke and dust in the air, and cause precipitation.

## 3. Negative-ion Generation---Truly Auxiliary Medical Effect

According to the data released by World Health Organization, negative-ion is called "air vitamin", and an animal needs about 10 billion ecological negative-ions every day, while our living environment can only provide about 0.1-2 billion ones; when the negative-ion concentration in the air is up to 400 ones/cm<sup>3</sup>, it can satisfy our basic needs for survival; when the negative-ion concentration in the air is up to 1000-2000 ones/cm<sup>3</sup>, it can satisfy our basic needs of health; when the negative-ion concentration in the air is up to 5000-10,000 ones/cm<sup>3</sup>, it can improve our immunity and antibacterial ability; and when the negative-ion concentration in the air is up to 300,000-500,000 ones/cm<sup>3</sup>, it can improve our natural healing ability, and many diseases can be healed without medication. Negative-ion plays a significant role in improving the air quality of treatment room and improving and maintaining the health of animals.

AEOLUS UC-1801 ICU INCUBATOR provides a high concentration of negative-ion generator, and the following is the negative-ion test report:

MODEL NO	Project Number	Applicable specification and model
DC9-12V	1403140013	UC-1801
I Electrical Parameters		
Input voltage/frequency	DC12V	
Output voltage	At DC12V -0.5--4.5KVd.c. (-3.5KV DC $\pm$ 0.5KV DC )	
Anion concentration	At DC12V $\geq 6 \times 10^6$ PCS/cm <sup>3</sup> $\pm 10\%$	
Insulation resistance	Power line external box $\geq 20M \Omega$ (500V DC)	
Dielectric strength	Power line external box AC 1800V 60HZ 5mA 3S No breakdown	

Rated power	< 1W					
Input current	< 80mA					
Operating temperature range	-10℃~+60℃					
Operating humidity range	40%~65%					
Storage temperature	-20℃~+70℃					
Humidity resistance	In case of no electricity, no abnormalities occur after being placed for 72 hours at ambient temperature of +25℃ and relative humidity of 95%.					
High-temperature characteristics	In case of no electricity, no abnormalities occur after being placed for 72 hours at ambient temperature of +70℃					
Low-temperature characteristics	In case of no electricity, no abnormalities occur after being placed for 72 hours at ambient temperature of -20℃.					
Drop test	Starting at 100cm (39.37") from the ground, free fall onto the board, it will still function properly.					
II Principle of Operation						
The negative ion generator boosts low voltage to negative high voltage DC through the voltage booster circuit. DC high voltage from the tip of carbon brush is used to produce high corona and lots of high-speed electrons are released. Since electronics cannot exist in the air for a long time (the life of existing electrons is n S level), they will be captured by oxygen molecules (O2) in the air immediately, and then negative ions are formed; the working principle of which is the same as natural phenomenon (negative ions are produced when there is thunder and lightning).						
III Inspection Report Attached	Test Voltage/Frequency	DC12V	Temperature	32℃	Humidity	62%
Item Templet	Output Voltage (KV DC)	Anion Concentration ( × 10 <sup>6</sup> PCS/cm <sup>3</sup> )		Insulation Resistance MΩ	Dielectric Strength	
①	-4.45	6.66		OK	OK	
②	-4.56	6.03		OK	OK	
③	-4.35	6.21		OK	OK	
④	-4.36	6.11		OK	OK	
⑤	-4.13	6.24		OK	OK	
⑥	-4.23	6.54		OK	OK	
⑦	-4.09	6.33		OK	OK	
⑧	-4.15	6.83		OK	OK	
⑨	-4.36	6.31		OK	OK	
⑩	-4.21	6.61		OK	OK	
Test equipment: KEC-990 air ion concentration tester, HG3830 high voltage test bench						

The above test report shows that: the nearby negative-ion concentration of AEOLUS UC-1801- ICU.INCUBATOR is up to 6 million ones/cm<sup>3</sup> standard, which has reached the medical-level Negative-ion emission standard. It is truly air purifying, and is effectively helpful for the rehabilitation and recovery of pets.



#### 4. Sterilization Function ---- Free from Cross-contamination

A UVC ultraviolet ray is the one that truly has germicidal effect, and C ultraviolet of which is easily absorbed by DNA of organisms, especially, ultraviolet ray of 253.7nm is the best. UV sterilizer is a purely physical method of disinfection, which is characterized by simple and convenient operation, high efficiency, no secondary pollution, easy for management and automation etc.

AEOLUS UC-1801- ICU.INCBATOR uses ultraviolet sterilizer, which is characterized by strong radiation intensity, high stability and high transmittance to quartz glass tube. The transmittance is of  $\geq 87\%$  and the radiation intensity is at 253.7-254 $\mu\text{m}$  and remains stable. Furthermore, high brightness mirror sterilization reaction chamber design is adopted. Compared with similar products from abroad, the sterilization strength increases by 18%-27%.

---- Generally, the bactericidal action of ultraviolet ray on bacteria and viruses can achieve sterilization rate of 99%-99.9% in one to two seconds.

---- The broad-spectrum of ultraviolet sterilization is the highest, and it can efficiently kill almost all bacteria and viruses.

---- The ultraviolet sterilizer has no secondary pollution. No chemicals will be added in ultraviolet sterilization, thus, water and its surrounding environment will not be polluted secondly. Furthermore, it will not change any ingredients in the water.

---- The operation of ultraviolet sterilizer is safe and reliable: traditional disinfection technologies such as chloride or ozone, the disinfectants themselves are highly toxic and flammable substances, while there is no such potential safety hazard in ultraviolet sterilization system.

#### 5. Medical Atomization Treatment Function---- Making the Treatment More Convenient

AEOLUS UC-1801- ICU.INCBATOR is equipped with silent medical atomization function, and biomedical engineering research shows: when the atomized particles are about 10 microns, medicines can only deposit in the mouth and throat, when the atomized particles are about 3-6 microns, can enter into the bronchus, and when the atomized particles are less than 2 microns, medicines can enter into the finest bronchus and alveoli. The atomized particles of medicine produced by AEOLUS UC-1801- ICU.INCBATOR are 0.5-4 microns. Meanwhile, intelligent control quantity of atomization is provided for your selection, and the maximum atomizing volume is up to 0.5ml/min. The atomization in the ICU is even, and the inhalation effect of atomized medicines is obvious. The medical built-in atomizer is characterized by low noise, which complies with pet ICU noise standard.

AEOLUS UC-1801- ICU.INCBATOR can also provide an external air compressor atomizer you are your selection according to users' preferences and usage, which offers greater choice space.

#### 6. Carbon Dioxide Concentration Monitoring Systems----A Defense Line to Guard Life

An increase in carbon dioxide concentration is accompanied by a decrease in oxygen concentration. Being anoxic under high concentration of carbon dioxide for a long time will cause pulmonary edema, cerebral edema, metabolic acidosis, electrolyte imbalance, shock, hypoxic encephalopathy, etc. When the carbon dioxide concentration reaches 1000PPM above, it will cause dizziness, drowsiness and reduced resistance to pets; when the carbon dioxide concentration reaches 2000-5000PPM, it will cause mild nausea, poor breathing, rapid heartbeat and electrolyte imbalance to pets, and seriously affect the health of pets and their postoperative rehabilitation; and when the carbon dioxide concentration is over 5000PPM, it will cause severe hypoxia, coma and even death.

The ICU of AEOLUS UC-1801- ICU.INCBATOR is equipped with a precise carbon dioxide concentration monitor, which can monitor changes in carbon dioxide concentration in the ICU. When being hypoxic state that concentration exceeds the standard value (exceeds the set value), and cannot be reduced below the

set value for three minutes, It will trigger the alarm system and start the intelligent ventilation system at the same time, being ventilated with the outside air and thus improve the air quality of ICU, which becomes the most important life defense line for pets.

#### 7. ICU Illumination Function----Creating Comfortable Therapeutic Environment

AEOLUS UC-1801- ICU.INCBATOR uses ten-level LED lights adjustment, which can simulate various scenes similar to parent, forest oxygen bar, warm home, forest dawn and dusk twilight, etc., providing a cozy and conformable therapeutic environment for pets.

#### 8. Setting of Security Mechanism---- No Worries in Use

AEOLUS UC-1801- ICU.INCBATOR adopts intelligent alarm system, and is equipped with corresponding fault display function, such as extreme temperature and excessive moisture alarm, lack-of-water alarm and main components fault warning lights.

AEOLUS UC-1801- ICU.INCBATOR is equipped with intelligent switch power keys, dividing into three controls, i.e. power-on, start and power-off. This design is energy saving and safer to use.

#### 9. Human Oriented Structural Design ---- Reasonable Operation, Clean and Convenient

All equipment components of AEOLUS UC-1801- ICU.INCBATOR adopt integrated design. All components are integrated in a control frame, and completely separated from the control circuit. Only by disassembling the control rack, the whole machine can be disassembled to clean various components, including fan, medical devices, heat sink and filter etc. The operation is extremely convenient, easy for troubleshooting, maintenance and equipment cleaning.

AEOLUS UC-1801- ICU.INCBATOR is equipped with two medical device import Interface channels, thus external medical equipment (such as oxygen machine) can be connected into the ICU, meanwhile, the side access port adopts medical valve importing design, being convenient for the importing of pets injection and infusion needles and relative sealing and fixing.

## Product Operation Instructions

#### 1. Start-up and Shutdown

a) One end of the power cord is plugged into the terminal board at the bottom support of the right side of the machine, and the other end into the socket of 220V/50Hz with reliable grounding. See fig. 7.1.2.

b) The power switch is located at the batter brace of the left side of the machine. See fig.7.1.2. It is the on/off switch of AEOLUS UC-1801- ICU.INCBATOR. When it is locked, all other keys are void; when it is unlocked, all other keys are valid.

c) The lock indicator lamp is at the on/off switch. When it is on, it means unlocked; when it is off, it means locked.

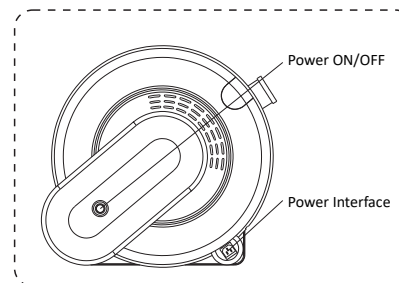


Figure 7.1.2

## 2. Temperature display

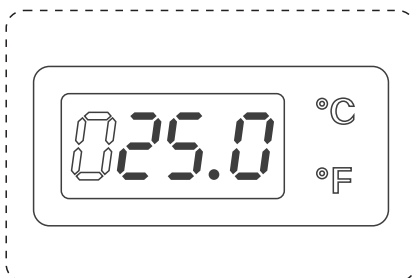


Figure 7.2.1

Fahrenheit/centigrade can be switched and displayed with a toggle key. The default temperature is shown in centigrade and in two-digit integer and one decimal place after which is a letter showing special indication at the time of setting the temperature parameter. Switch the Fahrenheit/centigrade indicator lamp. When the indicator lamp displays "C", it means centigrade; when it displays "F", it means Fahrenheit. See fig. 7.2.1.

## 3. Humidity / CO<sub>2</sub> Concentration Display

Humidity / CO<sub>2</sub> concentration display can be switched with the humidity / CO<sub>2</sub> concentration key. The default is humidity display in two-digit integer and one decimal place. CO<sub>2</sub> concentration is displayed in four-digit integer. Switch the humidity / CO<sub>2</sub> concentration indicator lamp. When the Indicator lamp is "HU", it means humidity; when the indicator lamp is "CO<sub>2</sub>", it means CO<sub>2</sub> concentration. See fig. 7.3.1.



Figure 7.3.1

## 4. Temperature Control Part

Key control: press "SET" key plus temperature "+/\_" key at the same time to adjust the temperature set point, and regulate it by  $\pm 0.1$  stepping. See fig. 7.4.1. The factory setting is the set point of 20°C. It can be adjusted down to 15°C and up to 38°C;

Smart alarm function:

Alarm for beyond the maximum set point: the temperature and the upper dash are displayed alternately every three seconds with alarm and flashing temperature indicator lamp.

Alarm for beyond the minimum set point: the temperature and the lower dash are displayed alternately every three seconds with alarm and flashing temperature Indicator lamp.

Damaged temperature sensor: flashing "ERR" display and flashing temperature indicator lamp in the temperature display.

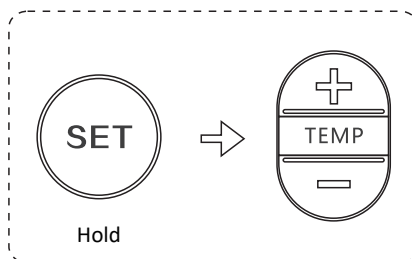


Figure 7.4.1

Control program:

- When the setting temperature is higher than the measured temperature, start the PID precise heating system and left/right fans. When the measured temperature is higher than the setting temperature, stop heating step by step.
- When the measured temperature is higher than the setting temperature, start the PID precise cooling system (need additionally purchased cooler) and the left fan to realize ventilation and air exchange inside and outside. When the measured temperature is lower than the setting temperature, stop cooling step by step and shut down fans after one-minute delay.
- When there is no additionally connected cooler, try to place an ice tube inside the atomizing chamber or at the sink as auxiliary cooling. The system will start the right fan and the left fan to realize ventilation and cooling.

Indicator lamp control:

Heating---the temperature indicator lamp is red

Cooling---the temperature indicator lamp is blue

Delay---the temperature indicator lamp is yellow

## 5. Humidity Control Part

Key control: press "SET" key plus humidity "+/\_" key at the same time to adjust the humidity set point, and regulate it by  $\pm 0.1$  stepping. See fig. 7.4.1. The factory setting is the humidity set point of 55%. It can be adjusted down to 40% and up to 65%.

Alarm function:

- Alarm for beyond the maximum set point: the humidity and the upper dash are displayed alternately every three seconds with alarm and flashing temperature indicator lamp.
- Alarm for beyond the minimum set point: the humidity and the lower dash are displayed alternately every three seconds with alarm and flashing temperature indicator lamp.
- Damaged humidity sensor: flashing "ERR" display and flashing humidity indicator lamp in the humidity display.
- Humidifier water shortage alarm (the water level sensor of the humidifier is disconnected): in this case, if the humidifier is started, the humidity Indicator lamp is off with water shortage alarm, and the humidity display shows "LAC". Meanwhile, it starts the smart water shortage protection system.

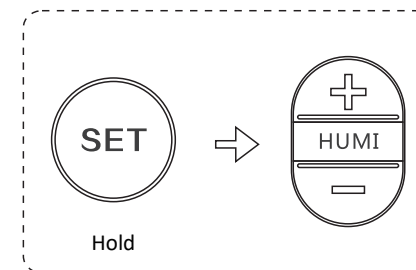


Figure 7.5.1

Control program:

- When the setting humidity is higher the measured humidity, start the humidifier and the right fan.
- When the measured humidity reaches the setting humidity, stop humidifying.
- When the measured humidity is higher than the setting humidity, start dehumidifying and the left fan; reversely start the right fan for air exhaust as auxiliary dehumidifying. If an additionally purchased cooler is connected, start the cooling function for dehumidifying.

Indicator lamp control:

The humidity indicator lamp is always on at the time of humidifying.

The humidity indicator lamp is flashing at the time of dehumidifying.

## 6. Anion Control Part

Press "anion" key to switch it ON; and press it again to switch it OFF.

When the anion is open, the anion indicator lamp is on; when the anion is closed, the anion indicator lamp is off. See fig. 7.6.1.

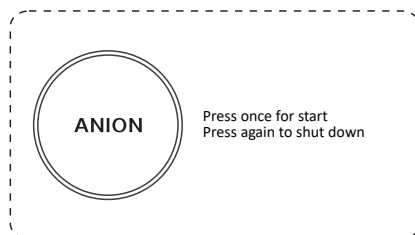


Figure 7.6.1

## 7. Medical Atomization Control

The atomizing volume of the medical atomizer has two grades, measuring by time and quantity. Press "medical atomization" key to start the atomizing function (it is the higher grade with the atomizing volume of 0.375ml/min and the auto-timing of 10 minutes). Press "atomizing volume regulation" key to switch between the higher grade and the lower grade of atomizing volume. The lower grade is 0.5ml/min with the auto-timing of 20 minutes. When the medical atomization is started, the atomization indicator lamp is flashing quickly (correspondent to the higher grade). Press "regulation" key to adjust it to the lower grade at the time of which the indicator lamp is slowly flashing. Press "medical atomization" key again to stop atomization, and the atomization indicator lamp is off.

a) Press "atomization" key to start the higher grade; press atomizing volume regulation key "+". To adjust it to the lower grade, press atomizing volume regulation key "-". See fig. 7.7.1.

b) Press "atomization" key and get into the medical atomization status (higher grade) during which the humidity display and control are shielded. The humidity display shows accumulated atomization time counting (in seconds). After atomization and 10 sec delay, it automatically exits the atomization mode and reverts to the humidity control mode.

c) During atomization, press "atomization" key to manually stop the atomization mode and revert to the humidity control mode.

d) In the atomization mode, the humidity control is not effective. Therefore, when the humidity exceeds the maximum set point, there is no alarm. If the humidity still exceeds the maximum set point in three minutes after it exits (closes) the medical atomization mode, the alarm is started.

Atomizer water shortage alarm: when the water tank or the medicine cup of the atomizer is short of water, stop atomization. The humidity display shows "ELAC" and the atomization lamp is off with water shortage alarm. Press "atomization" key again to enter the "atomizer off" status and recover humidity display.

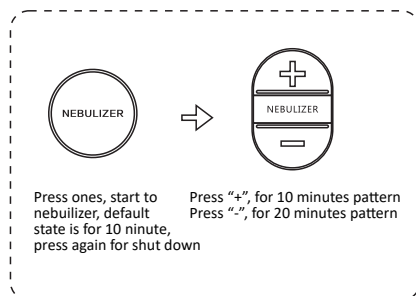


Figure 7.7.1

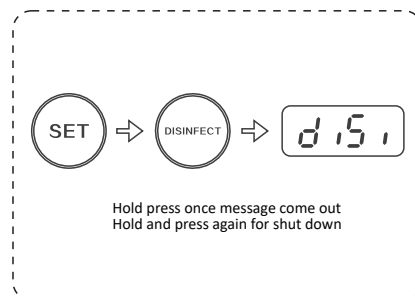


Figure 7.8.1

## 8. Disinfection and Sterilization Control Part

a) On: before starting the sterilization function, make sure that there is no pet in the ICU and the ICU door is closed. Meanwhile, press "SET" key plus "sterilization" key to enter the disinfection mode. See fig. 7.8.1.

b) Operation: under this mode, all other function keys are void except "SET" key and "sterilization" key. Both temperature display and humidity display show "DISI" (abbreviated disinfection); under the disinfection mode, switch on the UV disinfection lamp. There is violet light inside the ICU (with weak acrid scent). Meanwhile, start the left fan and the right fan to effect ventilation inside.

c) Off: press "SET" key plus "disinfection" key at the same time again to switch off the UV disinfection lamp, the disinfection indication lamp and the left fan to exit the disinfection mode and recover the normal work mode. At this time all function keys are valid except the disinfection key.

Note: when the ICU is under the normal work mode, it is invalid to press "disinfection" key individually.

## 9. Illumination Intensity Control

Press illumination adjustment key "+/-" to adjust ten grades of illumination intensity. See fig. 7.9.1. The highest is grade nine and the lowest is closed. Under the Initial start-up status, it is closed.

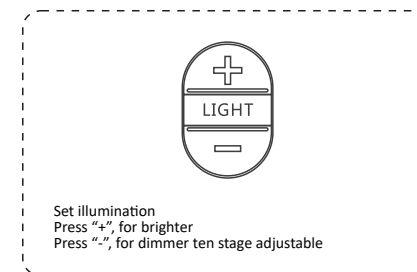


Figure 7.9.1

## 10. CO<sub>2</sub> Concentration Control

Key control: press "SET" key for five seconds to enter the parameters setting status. The humidity/CO<sub>2</sub> display shows flashing digits of CO<sub>2</sub> concentration set value. When the indicator lamp "P" (CO<sub>2</sub> concentration) is flashing, it is able to adjust the set value of CO<sub>2</sub> concentration alarm. Now press humidity "+/-" key to adjust CO<sub>2</sub> set value by ±10 stepping. The default factory setting is the alarm set point of 800PPM which can be adjusted down to 500PPM and up to 2000PPM. See fig. 7.10.1.

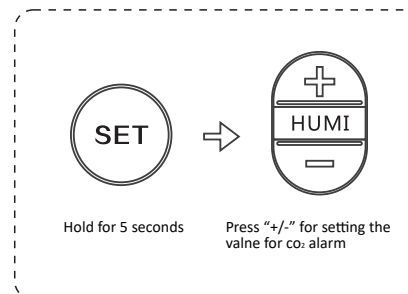


Figure 7.10.1

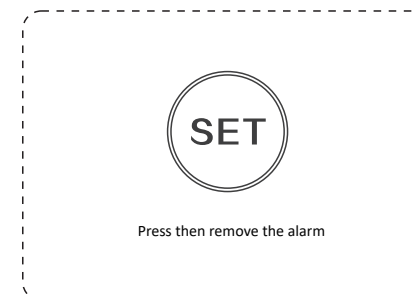


Figure 7.11.1



**Alarm and Processing Control:** when the CO<sub>2</sub> concentration is higher than the set point and remains constant for some time (to avoid false alarm for an abrupt increase of partial concentration), the CO<sub>2</sub> Concentration Indicator Lamp is flashing with the alarm. Start both the left fan and the right fan to enhance ventilation inside and outside. If the CO<sub>2</sub> concentration remains unchanged, open the ICU door to enhance ventilation. When the CO<sub>2</sub> concentration is equal to or lower than the set point, switch off the CO<sub>2</sub> indicator lamp and close the alarm. If fans are not needed when other functions are working the fans automatically stop.

**CO<sub>2</sub> Concentration Control:** the default factory setting is 1000PPM-1000 in a million (0.10%). PPM=solute mass/solution mass\*1000000. Reference set point: the CO<sub>2</sub> concentration of 0.1% is harmful to animals; the CO<sub>2</sub> concentration of 0.5% causes loss of breath and dizziness; the CO<sub>2</sub> concentration of 1% makes it increasingly difficult for animals to breath, gradually fading to a fatal point.  
CO<sub>2</sub> concentration in air: 0.030% – 0.039% (oxygen concentration: 20.95%).

## 11. Alarm Removal Key

Press "SET" key to remove disengage alarms. See figure. 7.11.1.

# Cleaning and Maintenance

1. Disinfection & Sterilization: after incubation or caring of one pet finishes treatment and care, it is suggested to disinfect and sterilize this machine. Do this once every week when not in use. It's important to maintain clean equipment.

2. Cleaning of the Humidifier (Old version): if the Humidifier falls to atomize, or the water spills out, take the following measures:

- Open the cover plate of the right Control Rack.
- Remove the water tank of the humidifier. (Remember to disconnect the plug on the water tank beforehand).
- Brush off the mineral substance such as water scale or verdigris on the Water Level Sensors (the three copper inserts on the inner surface of the water tank).
- Place the water tank back, insert the connection plug, and secure the cover plate.
- Brush off minerals on the inner water level sensor (the only slim metal column inside the humidifier chamber) and the bottom vibrator (water scale) by sticking a brush into the atomizing inlet.
- Cover the humidifier pipe.
- Cover the inner cover plate. Make sure the rubber at the pipe outlet covers two feet at the bottom of the cover plate. Installation tip: push the cover plate from top to bottom and then from outside to inside. Install screws into the cover plate.

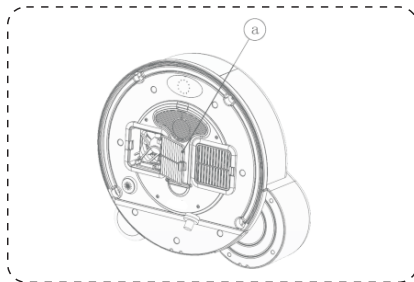


Figure 8.2.1-a

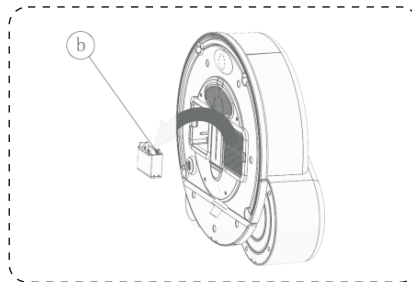


Figure 8.2.1-b

Only for the products sold before 2018.

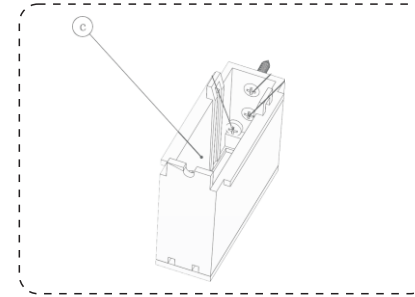


Figure 8.2.3-c

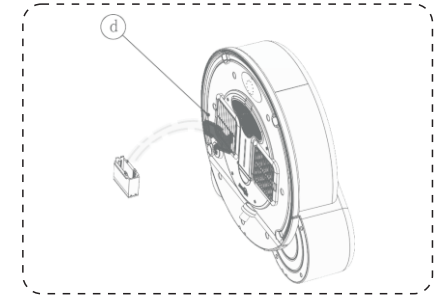


Figure 8.2.3-d

Only for the products sold before 2018.

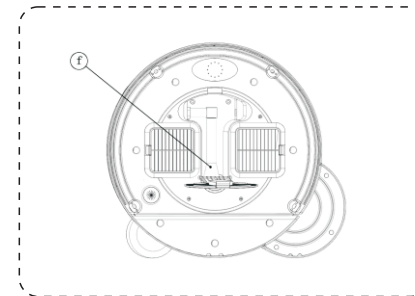


Figure 8.2.3-f

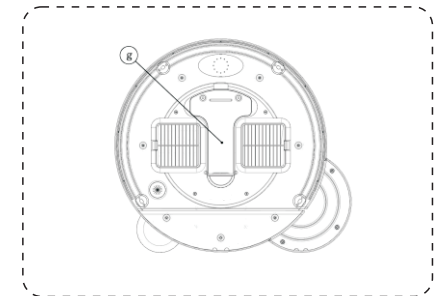


Figure 8.2.3-g

**Cleaning of the Humidifier (New version):** if the Humidifier falls to atomize, or the water spills out, take the following measures:

- As shown In Figure 8.3.3-a, Open the cover plate of the Water Vessel of the humidifier (on the control rack).
- As shown In Figure 8.3.3-b remove Humidifier from the Water Vessel and as shown In Figure 8.3.3-c, remove Water Level Sensors.
- Take out the Water Vessel of the humidifier as shown In Figure 8.3.3-d.

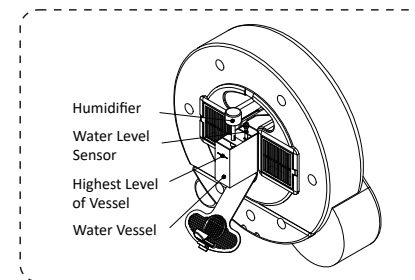


Figure 8.3.3-a

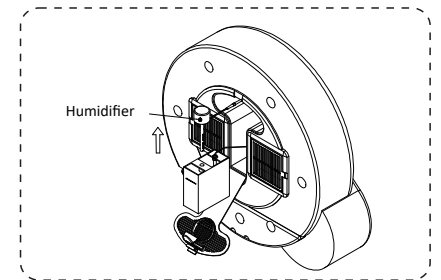


Figure 8.3.3-b

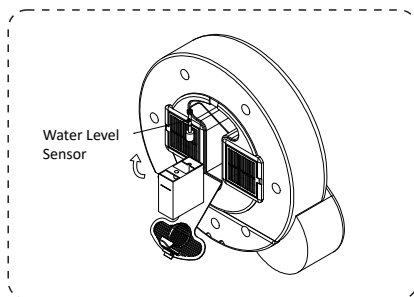


Figure 8.3.3-c

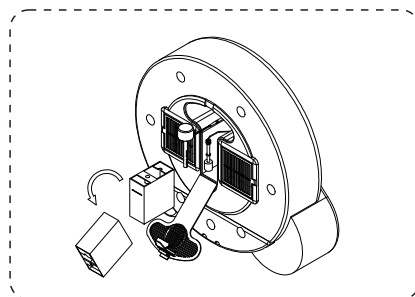


Figure 8.3.3-d

3. Complete Cleaning and Maintenance: remove and clean parts and assemblies periodically. Desired results are achieved only by cleaning after removal.

Steps for Disassembling Components:

Remove four screws from the master control rack on the right to take down the right master control rack, (remove the left master control rack in the same way). See figure. 8.3.1.

Remove power connection plugs on power cords on one side of the master control rack one by one. Note that plugs have different colors for each electrical component.

Warning: do not remove control racks or electrical components from control racks immediately after shut down; give it some time to cool down. The heaters in control racks reach high temperature. Remove control racks or contact heaters after a 30 minutes respite after shutdown.

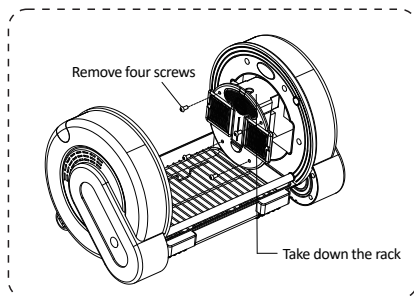


Figure 8.3.1-a

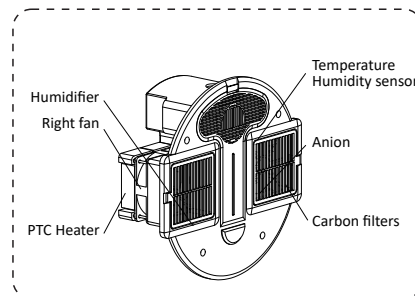


Figure 8.3.1-b

Remove four screws in the fan box and take out the whole heater. Clean the fan. Try not to remove heat sinks from the fan, especially PTC heater bands that cannot be removed from heat sinks. See figure. 8.3.2.

Remove and clean filters behind the square plastic section inside the chamber.

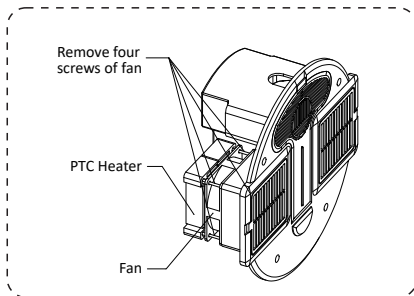


Figure 8.3.2-a

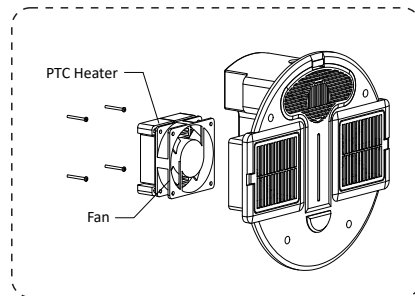


Figure 8.3.2-b

#### 4. Cleaning

The ICU must be empty at the time of cleaning. Cleaning is feasible only after removal.

#### 5. Maintenance

It is suggested to maintain the machine every two months during normal operation, which shall be carried out in accordance with removal, installation and maintenance procedures above.

## Trouble-shooting

Phenomenon	Cause analysis	Solution
After the power switch is pressed, there is no reaction or the power indicator lamp is still off	1.Bad power connection and plug connection 2.Damaged switch	1. Check power connection 2.Replace the power switch
Little atomizing volume form the medical atomizer	1.Too big atmizing particles of solution causes little atomizing volume 2.Solution contains syrup or colloform 3.Tablets are used for dissolution 4.Oily impurities on the surface of solution	1.Too big particles are not suitable for operation of this atomizer 2.When solution contains syrup or colloform,it requires with dry filter paper 3.When tablets are used for dissolution,dissolve sediments to remove impurites first 4.Remove oily impurities on the surface of solution with clean tissue firstivvv
No atomizing form the atomizer	1.Water shortage of the water tank of the atomizer 2.Damaged ultrasnic chip of the atomizer 3.Water scale in the tank of the atomizer	1.Add water into the water tank until it is at the same level of the cup bottom 2.Replace the ultrasonic chip 3.Gently scrub the inside of the tank with lint
Remaining solution not atomized in the cup after atomization of the medical atomizer	Atomizing time and volume depend on concentration of atomizing liquid	Repeat the atomization operation procedure if not completely atomized once

The humidity is not increased after raising the set point of humidity	1. Water shortage in the bottle of the humidifier 2. Damaged chip of the humidifier 3. Water scale in the metal water level sensor inside the humidifier tank	1. Add water into the bottle 2. Replace the humidifier 3. Remove water scale inside according to the chapter of "separate cleaning of the humidifier", or try to dip some vinegar into the water bottle of the humidifier
Operational keys on the panel do not work	Bad contact of panel plu-gins Damaged panel keys	1. Install plug-ins properly 2. Replace the panel
No heating	1. Failed or damaged fan 2. Heater (heat sink) with bad contact or damaged 3. ICU door open	1. Check and replace fan 2. Replace heater band or heat sink 3. Close ICU door
No cooling	1. No external cooler connected 2. Cooling pipe of the cooler not connected to the tank 3. Signal line of the cooler not connected to the machine	1. Connect it to an externally purchased cooler 2. Check the cooling pipe 3. Plug the signal line into the colling socket of the tank
Continuous alarm for CO <sub>2</sub> concentration	1. Failed or damaged fan 2. Not enough instant ventilation inside the chamber 3. Damaged CO <sub>2</sub> sensor	1. Check fans 2. Open the ICU door 3. Replace the CO <sub>2</sub> concentration sensor

# AEOLUS UC-1801 ICU.INCUBATOR

## Instructions on Maintenance

Dear user:

Thank you for using AEOLUS UC-1801-ICU.INCUBATOR. We provide following services as dictated in this warranty manual (and the invoice or other certificates of product purchase according to "Law of the People's Republic of China on Protection of Consumers' Rights and Interests", as well as "Provisions on the Liability for the Repair, Replacement and Return of Some Commodities" issued by the CSBTS and the Bureau of Finance.

1. One-year warranty for the whole machine.
2. Two-year warranty for key components (including medical atomizer, humidifier, anion generator, ultrasonic sterilizer, fan, tank, door, master control panel, PTC heating chip).
3. Six-month warranty (from date of purchase) for replacement of key integral parts (including medical atomizer, humidifier).
4. Warranty services are not provided under the following situations, (but paid repair services are available for the life time of the equipment).

[A] No warranty card or any other certificate of product purchase presented

[B] Altered certificate of product purchase

[C] Damage caused by accident, abuse or misuse

[D] Repair privately without authorization of our company

[E] Products can be used after repair beyond the valid period of the three warrantees

5. The valid period of the three warrantees is from the date of product purchase, not including time of repair and waiting for components if out of stock. During the valid period of three warrantees, consumers can have products repaired, replaced or returned with invoice or certificates of product purchase.

6. Within 15 days after purchase, consumers can return, replace or repair products if there is any performance failure found.

7. Within 30 days after purchase, consumers can replace or repair products if there is any performance failure found.

8. "Performance failure" indicates that products cannot meet safety and hygiene requirements with unreasonable danger threatening safety of human body or properties; or products do not have complete functional performance as required, or cannot meet explicitly expressed quality conditions.

Advance replacement warranty service:

Your warranty service may involve advance replacement warranty service. If the hardware is found to be defective after diagnosis, we will send replacement parts directly to you. After receiving the replacement, you must return the defective part to us within the specified period (normally five days). The transportation and insurance fees rising from return of the defective part needs to be paid by you. If you cannot return the defective part, we will ask you for replacement fees. We provide replacement services during normal working hours, 8:00 a.m. to 5: 00 p.m. Monday to Friday. The office time may be different according to local time zones.

## Mail Warranty Service

Your warranty service may involve mail or courier. According to mail service provisions, you must send defective hardware to an authorized maintenance facility via freight collect. We will repair and return it to you.

Door-to-door pickup and return warranty service:

Your warranty service may involve door-to-door pickup and return services. The company or a designated repair point will send technical employees to pick up defective hardware at your location and send it back after repair. For such services, the company pays all maintenance, transportation and insurance fees.

User Instructions:

For products beyond the warranty period, either the company sends service employees to pick up or you can mail the defective part to the maintenance point. We provide our repair or replacement services and parts (fees for maintenance after warranty) covering shipment costs, labor costs for repair and component cost. Users can refuse to pay if maintainers cannot comply with unified charging standards of the company.

Contact Customer Support and Service:

If information provided in these user instructions cannot solve your problem, contact the customer support of AEOLUS Company at this website:

[www.aeoluspet.com](http://www.aeoluspet.com)

Here you can:

1. Communicate with our technical employees online.
2. Use English if technical support is not available in some specific language.
3. Send an email to the customer support of Aeolus;
4. Find the global/domestic customer support telephone number of Aeolus
5. Find Aeolus service center.